EZ-View® Auto Light Flat Screen Viewer

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Indications for Use

This product is intended to view dental radiographs.

Contraindications for Use

None known.

Warnings

- Not recommended for use with wet films.
- TO DISCONNECT POWER, UNPLUG THE POWER ADAPTER.
- **DO NOT ATTEMPT INTERNAL SERVICE.**
The interior of the viewer is only accessible by removing hardware with tools and should only be opened and serviced by an authorized technician. Since the interior of the unit contains high voltage components, failure to heed this warning may result in equipment damage, personal injury and/or death. Please call Rinn Customer Service with any questions, 1-800-323-0970 or 1-847-742-1140.
- **DANGER: EXPLOSION HAZARD — DO NOT USE IN THE PRESENCE OF FLAMMABLE ANESTHETIC MIXTURES WITH AIR, OXYGEN OR NITROUS OXIDE**
- Lamps cannot be replaced by the customer. See "DO NOT ATTEMPT INTERNAL SERVICE", above, for applicable warnings.
- USE ONLY RINN-SUPPLIED POWER ADAPTER.
- Rinn viewers utilize fluorescent lamps (bulbs). Fluorescent lamps contain mercury. Dispose of according to local, state or federal laws.

Initial Set-up

Remove unit from packaging.

1. Plug power supply into outlet.
2. Plug power supply into viewer.

PLEASE NOTE: Your new EZ-View Auto Light Viewer includes a Universal Film Clip Roller feature, allowing several individual rollers to "float" — allowing placement of multiple mounts and films of different thicknesses to be inserted across the top of the viewing area. If you shake your viewer, they will rattle; this is normal and does not require repair.

Desk Mounting

1. Remove desktop leg from rear of unit.
2. Insert leg into one of three positions available (15°, 25°, or 75°).
3. Place on stable countertop or other flat surface.

Wall Mounting

1. Mark position for mounting screws 10⁵/₁₆" (270mm) apart.
2. Drill holes with 5/₁₆" (8mm) drill and insert anchors.
3. Insert screws. Tighten to within 1/₈" (3mm) of wall.
4. Align keyhole slots on viewer with screws. Slip on and push down to secure.

Magnifier

If you have the optional 2X magnifier, it can be easily snapped into the top and bottom groove of the viewer. Place the frame into the bottom groove, then pull up and snap it into the top groove.

Operation

Manual On – Manual Off

- Press either the + or the – push button switch to turn the unit on.
- Press both the + and – push button switches simultaneously to turn the unit off.

Auto-light On – Auto Off

- Insert mount or film at the rectangle in the center of the universal film clip.
- Remove mount or film from the universal film clip and unit will automatically turn off after five (5) minutes.

Note: Unit will not automatically turn off unless a mount or film has been inserted at the rectangle in the center of the universal film clip and then removed. If unit is manually turned off with a film in the rectangle in the center of the universal film clip, press either the + or the – push button switch to turn the unit on.

Brightness adjustment

- Press the + push button switch to increase brightness.
- Press the – push button switch decrease brightness.
- Press and hold either push button switch for fast adjustment.
- Press and release for finer, stepped adjustments.

Note: The unit takes approximately 30 seconds to reach full brightness.

Maintenance

- Always disconnect power to the viewer before cleaning or servicing.
- As needed, clean with a soft, damp cloth. DO NOT USE AN ABRASIVE CLEANER.
- Magnifier lens should be cleaned with lens tissue or a soft cloth.

Troubleshooting

If viewer does not light:

1. Check that power adapter is completely inserted into wall receptacle and into power connection on unit.
2. Check that mount is fully inserted at the rectangle in the center of the universal film clip for auto-light function.
3. Check that dimmer is not at the lowest setting, press and hold the + push button for 5 seconds to brighten.

Please call Rinn Customer Service with any questions, 1-800-323-0970 or 1-847-742-1140.

DO NOT ATTEMPT INTERNAL SERVICE

The interior of the viewer is only accessible by removing hardware with tools and should only be opened and serviced by an authorized technician. Since the interior of the unit contains high voltage components, failure to heed this warning may result in equipment damage, personal injury and/or death. Please call Rinn Customer Service with any questions, 1-800-323-0970 or 1-847-742-1140.

Specifications

Power Requirements

For all units: 90VAC to 264VAC, <0.5A RMS at 90 VAC input

Power consumption 12VDC, Class II
UL 60601-1

USE ONLY RINN-SUPPLIED POWER ADAPTER.

Viewing Source

Two long-life 4-watt cold cathode fluorescent lamps. 10,000 hours of life.

Environmental Conditions

- For indoor use only, and in temperatures of 32°F to 104°F (0°C to 40°C). Altitude up to 2000 M.
- Maximum relative humidity 80% for temperatures up to 31°C, 50% for temperatures up to 40°C.
- Overvoltage Category II
- Pollution Degree II

Replacement Parts

067404	110V US Power Supply
067405	220V Euro Power Supply
067103	Desk Top Leg
067601	Magnifier Assembly
3207	Magnifier Lens

Lamps are not customer-replaceable.

Warranty

Disassembly of unit voids warranty.

DENTSPLY RINN warrants this product to be free of defective materials and workmanship for one year from date of purchase when used in accordance with the operating instructions furnished herewith. This warranty is limited to the original purchaser. Any component that our examination reveals to be defective shall either be repaired or replaced at our option. There are no other warranties, either expressed or implied. Damages are limited to repair or replacement of product. DENTSPLY RINN expressly disclaims liability for incidental and consequential damages resulting from use of its product. DENTSPLY RINN neither assumes, nor authorizes any person to assume for it, any other liability in connection with the sale or use of its product.

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**A Division of
DENTSPLY International, Inc.**

1212 Abbott Drive
Elgin, IL 60123-1819
847 742 1115
Customer Service: 800 323 0970
FAX (24 hours a day): 800 544 0787
www.rinncorp.com

International:
Phone: +1 847 742 1140
Fax: +1 847 742 1122

Contacts:

Worldwide Sales Manager:
Richard Stagg 847 742 1115 x180
rstagg@dentsply.com

Eastern U.S.:
Edward Mileto 631 475 3763

Central U.S.:
Lawrence Zeno 773 935 5864

Western U.S.:
Jeff Furgerson 760 732 1121

Europe:
Luc Demeester 1+ (32) 1552 9826

Germany:
Gisela Rottmann 1+ (49) 6306 1280

DENTSPLY Australia Pty. Ltd.
11-21 Gilby Road
Mount Waverly, Victoria 3149, Australia

**DENTSPLY LAR
DENTSPLY Industria e Comercio Ltda.**
Rua Alice Herve, 86
Bingen
25665-010 Petropolis, RJ, Brazil

DENTSPLY Canada Ltd.
161 Vinyl Court
Woodbridge, Ontario
Canada L4L 4A3

DENTSPLY France SAS
4, rue Michael Faraday
78180 Montigny le Bretonneaux, France

**DENTSPLY Germany – Konstanz
DENTSPLY De Trey GmbH**
De-Trey Strasse 1
D-78467 Konstanz, Germany

DENTSPLY Hong Kong
23/F Pacific Link Tower
Southmark, 11 Yip Hing Street
Wong Chuk Hang, Hong Kong

DENTSPLY Italia Srl.
Via A. Cavaglieri, 26
1-00173 Rome, Italy

DENTSPLY Japan K.K.
Yushima Building
3-14-9, Yushima, Bunkyo-ku
Tokyo 113-0034, Japan

DENTSPLY Mexico, S.A. de C.V.
Calzada Vallejo No. 846
Col. Industrial Vallejo
02300 Mexico, D.F.

DENTSPLY DeTrey Sarl
Baar Office
Oberdorfstrasse 11
CH-6342 Baar, Switzerland

**DENTSPLY United Kingdom – Weybridge
DENTSPLY Ltd.**
Hamm Moor Lane
Addlestone, Weybridge
Surrey, KT15 2SE, England