1

Storage: Store at room temperature (18-25°C / 65-77°F).

4. Cycle steam autoclave at 273°F (134°C), 30PSI (206.8kPa). Do not exceed 270°F (132°C) for 10 minutes at

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Adverse Reactions: None known. Do not pull on sensor cord during use.

Precautions: Follow sterilization instructions carefully to prevent damage to plastic pieces during sterilization. Use

Warnings: Sterilize before use. DO NOT use phenol-based glutaraldehyde or

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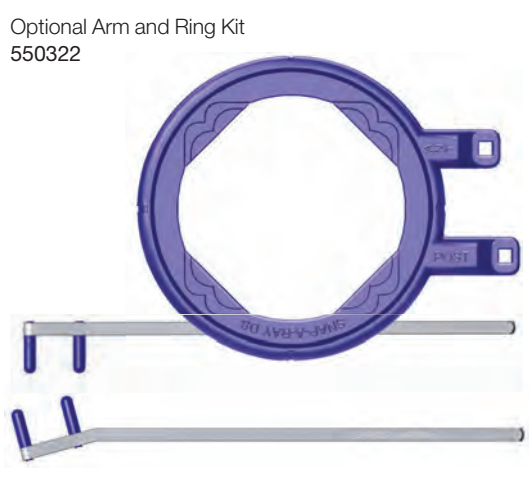
Manufactured by DENTSPLY RINN A Division of DENTSPLY International Inc. 1212 Abbott Drive Elgin, IL 60123 USA RINN Customer Service: 800-323-0970 USA only 847-742-1140 International www.RinnCorp.com

EC REP DENTSPLY Limited Building 1, Aviator Park, Station Rd. Addlestone, Surrey KT15 2PG UK

Snap-A-Ray® DS UNIVERSAL SENSOR HOLDER

Snap-A-Ray® DS

UNIVERSAL SENSOR HOLDER

DENTSPLY
RINN

Step-By-Step Instructions

1. Insert sensor into device as indicated **A** and slide locking mechanism **B** to secure sensor.
2. Use optional Snap-A-Ray positioning arm and ring **C** for alignment.
3. Position holder in patient's mouth.
4. Align x-ray tube.
5. Take exposure.
6. Adjust as needed for the next desired position.

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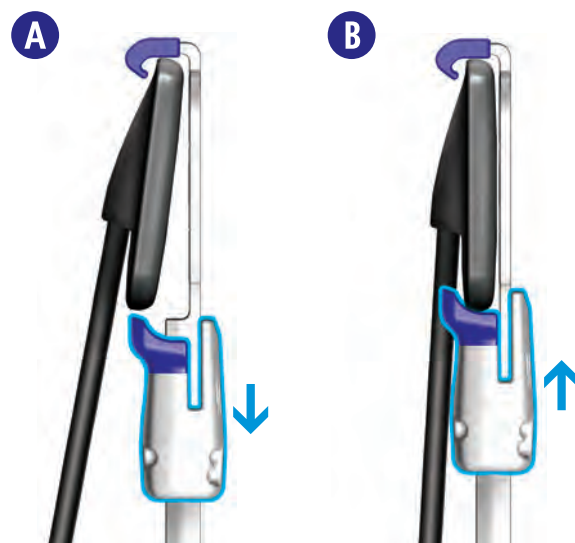
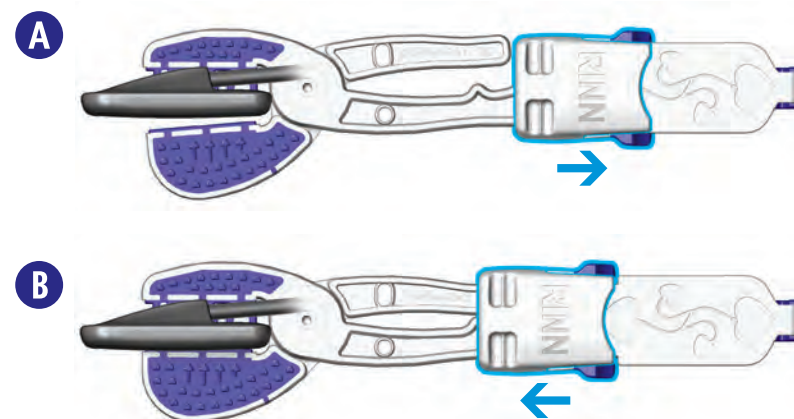
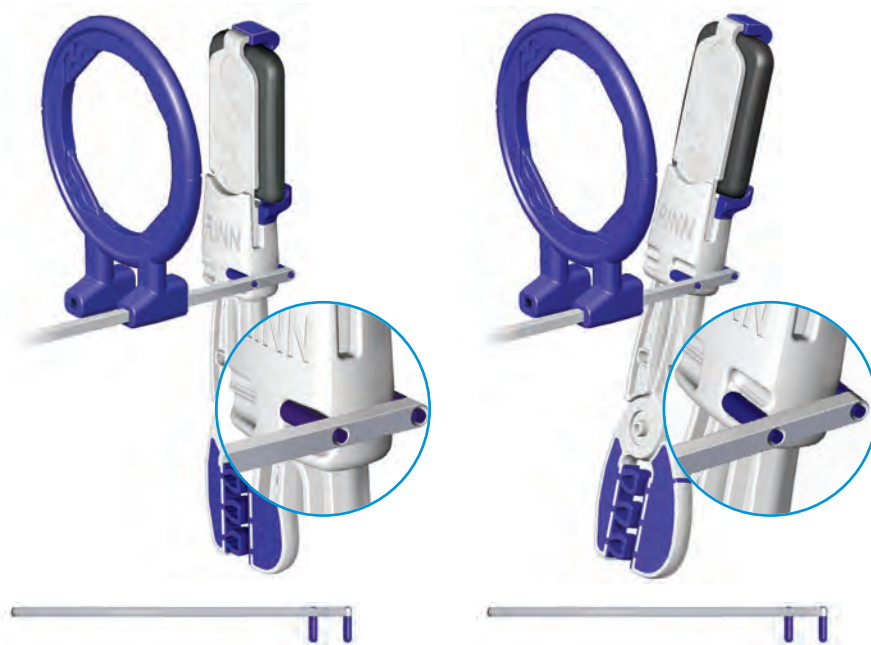
C Normal Palate
Normaler Gaumen
Palais normal
Palato normale
Paladar normal

Shallow Palate
Flacher Gaumen
Palais plat
Palato basso
Paladar poco profundo

To prevent slipping from loose sensor cover, grip off-center **1** and slide sensor to correct position **2**.

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