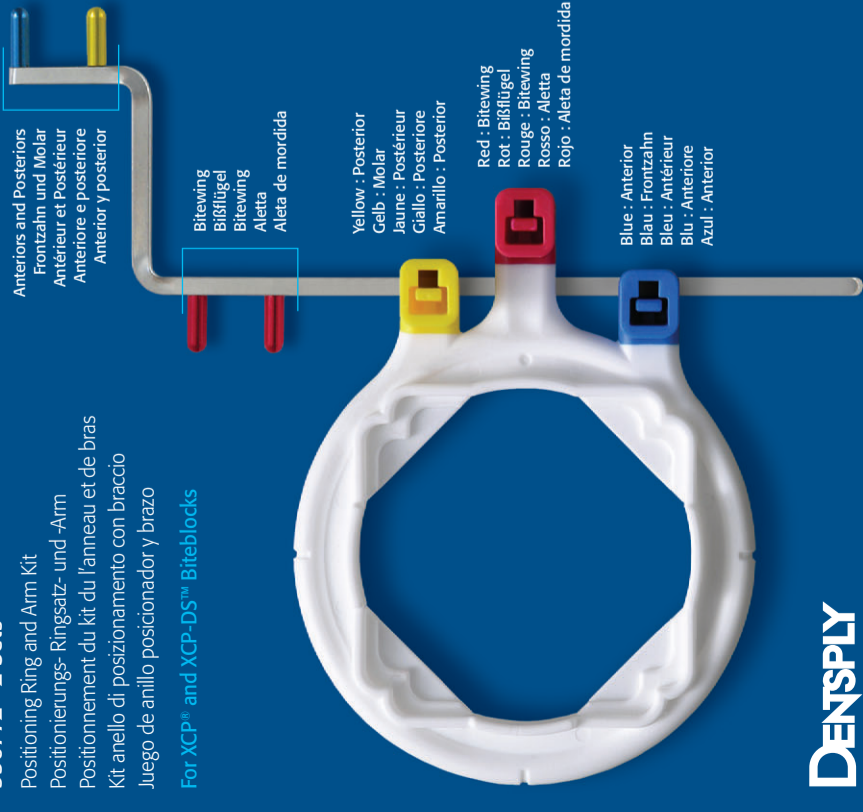
RINN XCP-ORA™

ONE RING & ARM POSITIONING SYSTEM


550771 1 Set
550772 2 Sets

Positioning Ring and Arm Kit
Positionierungs-Ringsatz- und -Arm
Positionnement du kit de l'anneau et de bras
Kit anello di posizionamento con braccio
Juego de anillo posicionador y brazo

For XCP® and XCP-DS™ Biteblocks

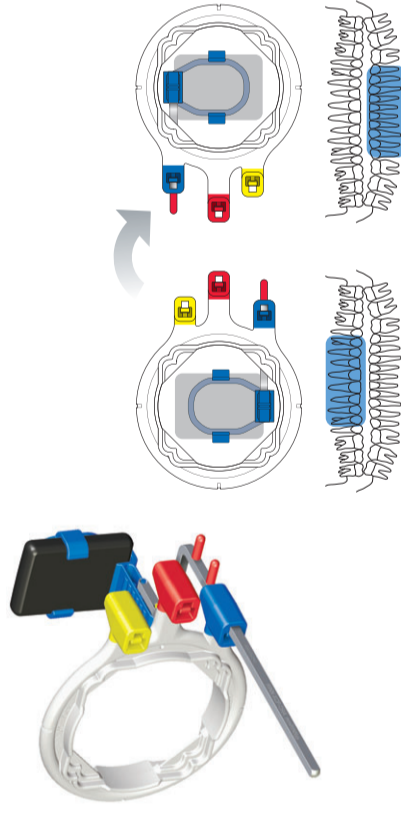Vertical Anterior Vertical Frontzahn Vertical Antérieur Verticale anteriore Vertical anterior


NOTE: Depending on palate, biteblock may flex for improved patient comfort. When this occurs, adjust vertical angulation of the tube head 5-10°.

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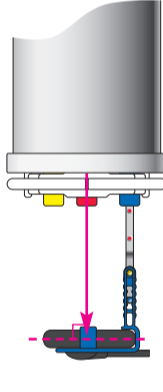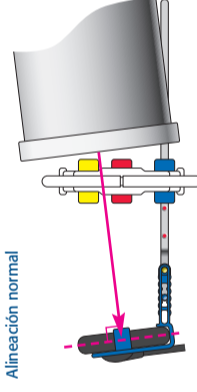
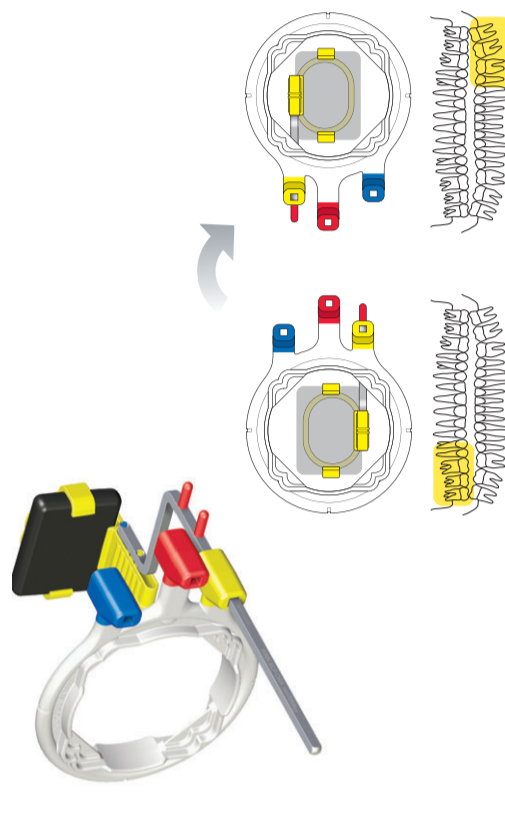
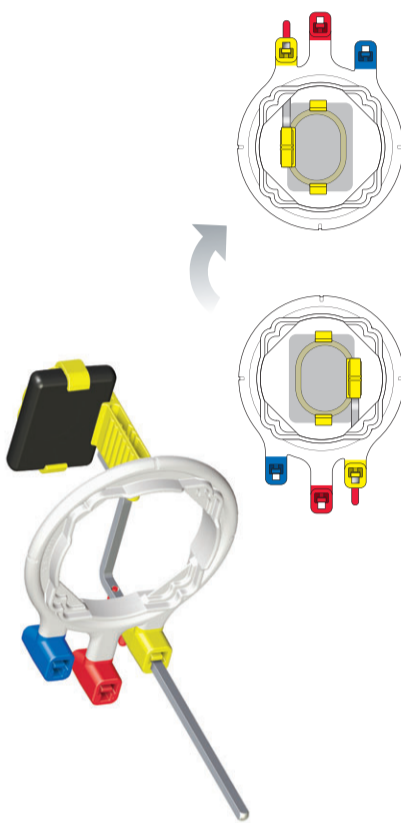
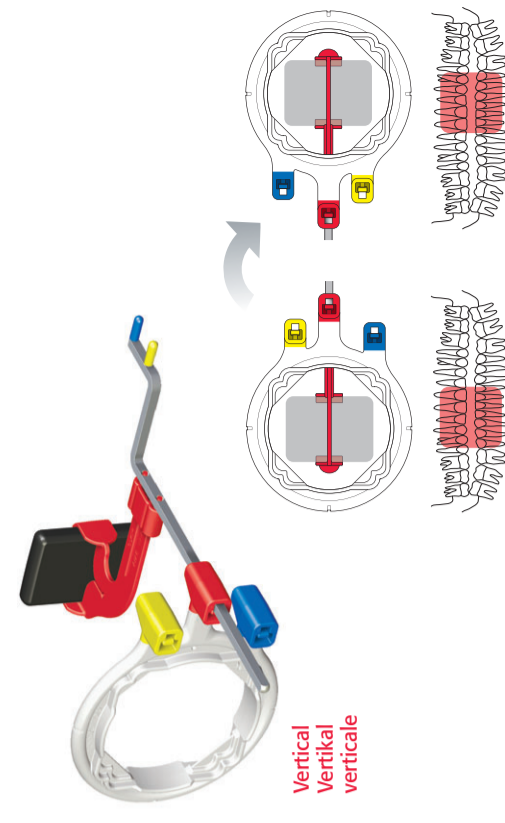
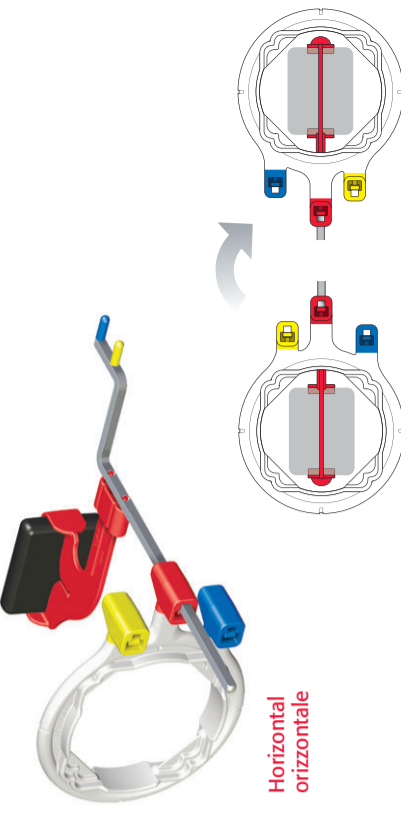
Bitewing Bißflügel Bitewing Aletta Aleta de mordida

Instructions

1. Insert XCP-ORA positioning arm into the appropriate color-coded channel of the aiming ring. NOTE: The anterior and posterior biteblocks both connect to the same positioning arm pins.
2. Attach Rinn XCP or XCP-DS compatible biteblock for film or digital sensor (sold separately).
3. Correctly assembled, the imaging area should be centered when viewed through the aiming ring.
4. Position holder in patient's mouth, slide ring close to patient's face, align x-ray tube and take exposure. NOTE: With digital sensors, place holder in the center of the patient's mouth for added patient comfort.
5. After taking radiograph, adjust for the next desired position.
6. When finished, disassemble the XCP-ORA arm and ring for steam autoclaving.

Cleaning and Sterilization

1. Disassemble instrument(s). **Separate the arm and ring for steam autoclaving.**
2. Remove debris from components with hot water and soap.
3. Put components in sterilization pouches, and place in the middle tray of the autoclave, away from autoclave walls and heating element. Plastic parts must be in a separate pouch from metal arms, to avoid melting or warping.

4. Cycle per autoclave manufacturer's instructions. Do not exceed 270° F (132° C).
- DO NOT use phenol-based glutaraldehyde.
- DO NOT use special application or ultrasonic cleaners.
- DO NOT CHEMIClave or DRY-HEAT STERILIZE.
- DO NOT cold sterilize.

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© 2009 DENTSPLY Rinn
RINN® and XCP® are registered trademarks, and XCP-DS™ and XCP-ORA™ are trademarks of DENTSPLY International, Inc.
Elgin, Illinois 60123-1819 U.S.A.