XCP-DS®

Digital Sensor Positioning System for
Accent, Actéon, Cyber Medical,
Cygnus, Dentrax, Dürr,
Gendex® VisualiX® USB / Gx-SUSB,
Progeny, Visiodent

005519 XCP-DS Instruction Poster

550593
Universal Horizontal
Universal Horizontal
Universal Horizontal
Universal Horizontal
Universale Orizzontale

550594
Universal Vertical
Universal Vertical
Universal Vertical
Universal Vertical
Universale Verticale

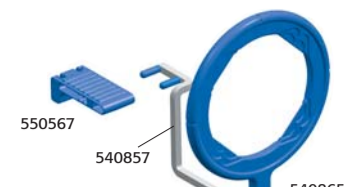
550595
Large Horizontal
Gross Horizontal
Grand Horizontal
Grande Horizontal
Grande Orizzontale

550596
Large Vertical
Gross Vertikal
Grand Vertical
Grande Vertical
Grande Verticale

Always use a barrier on the sensor.
Bitte immer Hygieneschutz verwenden.
Toujours placer un protecteur sur le capteur.
Coloque siempre una protección en el sensor.
Coprire sempre il sensore con apposita protezione.

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

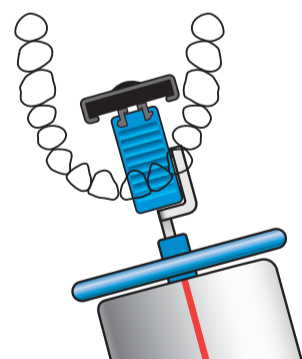
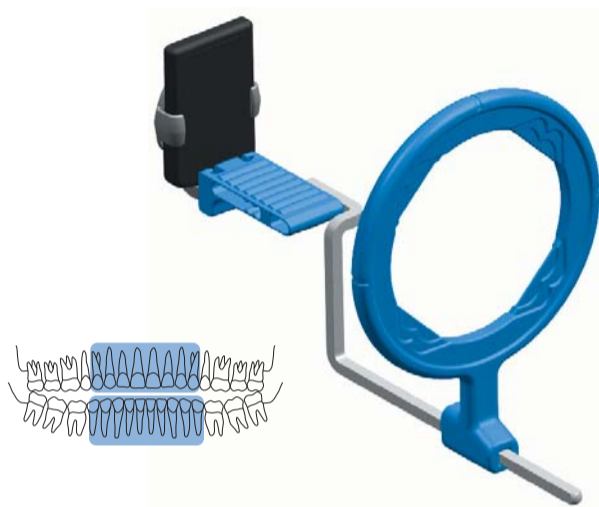
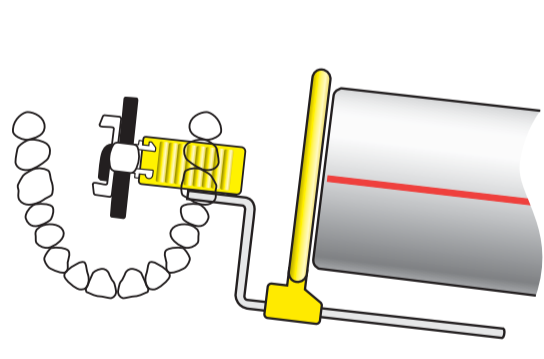
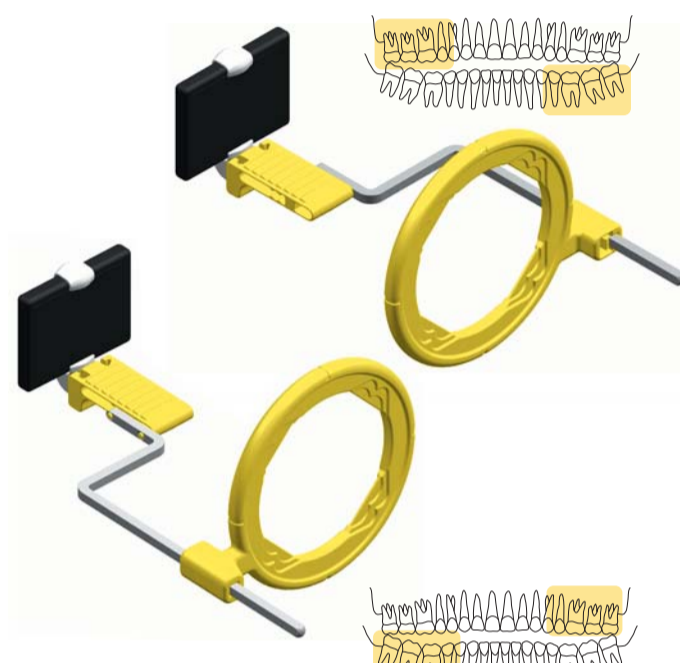
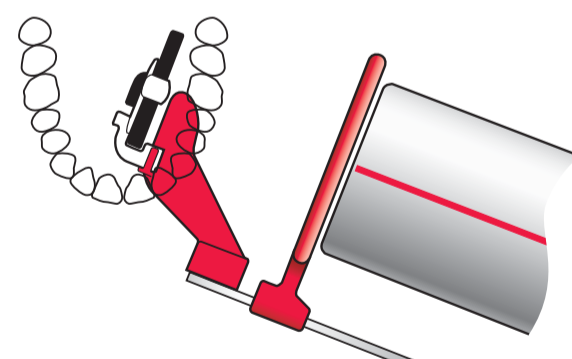
Endodontic
Endodontie
Endodontie

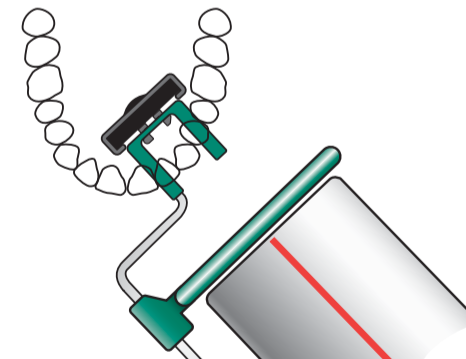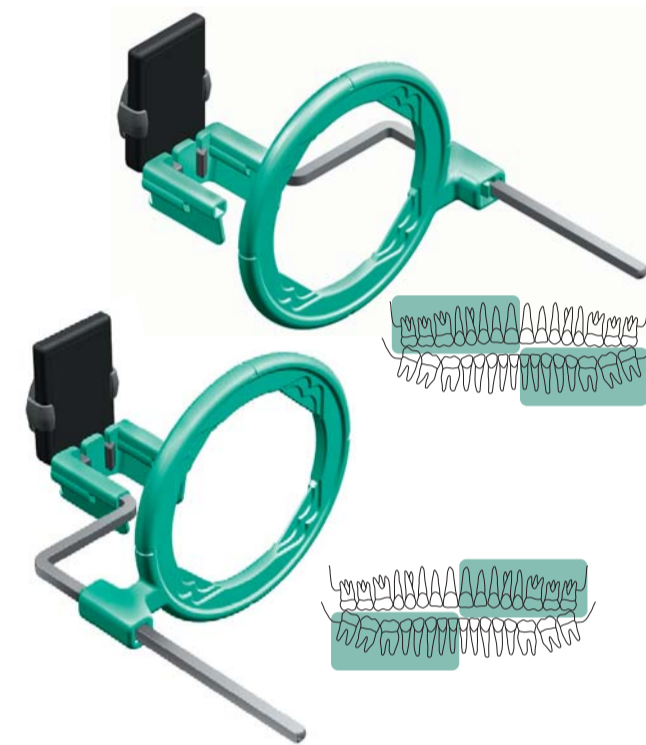Endodontia
Endodonzia

Cleaning and Sterilization Procedure

Prior to use, always cover the sensor with a disposable hygienic protective cover before inserting the sensor into the holder. Discard cover after use. After use with a patient, the parts of the positioning system must be cleaned and sterilized.

1. Disassemble instrument(s).
 2. Remove debris from components with hot water and soap.
 3. Put components in sterilization pouches, and place in the middle tray of the autoclave, away from autoclave walls and heating element. Plastic parts must be in a separate pouch from metal arms, to avoid melting or warping.
 4. Cycle per autoclave manufacturer's instructions. Do not exceed 270°F (132°C).
- DO NOT use phenol-based glutaraldehyde.
 - DO NOT use special application or ultrasonic cleaners.
 - DO NOT CHEMICLAVE or DRY-HEAT STERILIZE.
 - DO NOT cold sterilize.

Note: Plastic parts have a limited life and should be replaced periodically. Any method of sterilization will shorten the life of plastic parts.

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