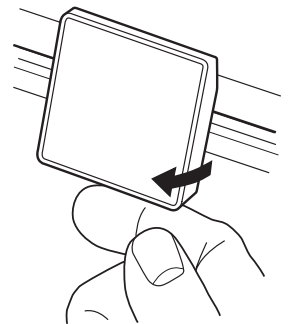
Slimline and Universal Duplicators

To open latch
Pour ouvrir le loquet
Öffnen der Verriegelung
Para desenganchar
Per aprire la chiusura

Please Note

- Before you begin, duplicating bulb must be removed from the foam, and foam packing between glass and diffuser removed.
- Do not latch when not in use.
- Exposure times will be 30-35 seconds, depending on duplicating film used and density of the original. To darken duplicate, decrease exposure time. To lighten duplicate, increase exposure time.

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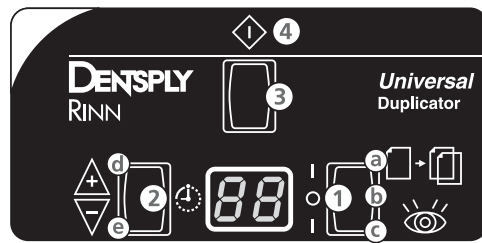
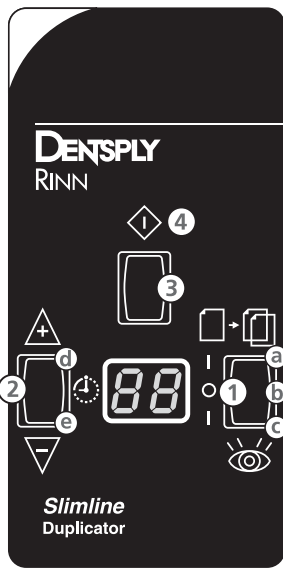
Initial Set-up

1. Remove unit from packing material.
2. Remove duplicating bulb from foam between glass platen and diffuser plate.
3. Remove foam pack between glass platen and diffuser plate.
4. Place on stable countertop or other flat surface.
5. Lift diffuser out with the pull tab adhered to the diffuser.
6. Insert bulb into the socket by pushing in and turning the bulb 1/4 turn.
7. Replace diffuser plate and peel off the pull tab. Replace glass platen.
8. Plug into grounded outlet.

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Operation

1. Turn on power.
2. With mode switch on "view", position films to be duplicated on the platen, raised dots facing down.
3. Set the mode switch to "Duplicate".
4. Set the timer to desired exposure time (30-35 seconds recommended).
5. In approved safelight conditions, place duplicating film emulsion side down over the film(s) to be copied.
6. Close cover and latch securely. (Leave cover unlatched when not in use).
7. Press the duplicate start button.
8. When cycle is done, remove the exposed duplicating film and process as you would radiograph film.

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Maintenance

Keep the platen free of dust and fingerprints that will effect the quality of the film duplicate. Dust with a soft, lint-free cloth or brush to avoid scratching the platen glass. The platen and diffuser are easily removed for cleaning.

Replacing Bulbs

Disconnect/unplug unit before replacing bulbs.

Remove platen glass and diffuser to access bulbs.

Duplicating Bulb: Twist the bulb a quarter turn and lift out. Insert fresh bulb and secure by a quarter turn.

Safelight Bulb: Unscrew the bulb completely and lift out. Insert fresh bulb and turn clockwise until secure.

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(too light for good diagnosis)

Cause: Over exposure.

Film has been exposed to light or is too old.

Remedy: Decrease exposure time. Handle and store film based on manufacturer's recommendations. Use fresh film.

Dense Image

(too dark for good diagnosis)

Cause: Under exposure.

Emulsion side of film is up.

Remedy: Increase exposure time. Place duplicating film emulsion side down.

Note: Duplicating film is the opposite of x-ray film. If the image is too light, decrease the exposure time; if the image is too dark, increase the exposure time.

For technical support on your Rinn duplicator, please call 1-800-323-0970 or 1-847-742-1115.

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