67-0400 120v / 60Hz / .18A
67-0420 220v / 50Hz / .18A
67-0425 240v / 50Hz / .15A

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TO ACCESS INTERIOR

To replace lamps or access mounting holes in back panel:
Slide diffuser out through slot in side of unit (1).

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DESK MOUNTING

1. Remove 2 screws from bottom of unit.
2. Position brackets (2). Attach with larger painted screws provided.
3. Apply pads.

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WALL MOUNTING

1. Mark position for mounting screws 7" apart.
2. Drill holes with 5/16" (8mm) drill and insert anchors.
3. Insert screws. Tighten to within 1/8" (3mm) of wall.
4. Align keyhole slots on viewers with screws. Slip on and push down to secure.

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DANGER: EXPLOSION HAZARD
DO NOT USE IN THE PRESENCE OF FLAMMABLE ANESTHETIC MIXTURES WITH AIR, OXYGEN OR NITROUS OXIDE.

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MAINTENANCE

Clean with a soft, damp cloth. DO NOT USE AN ABRASIVE CLEANER. Magnifier lens should be cleaned with lens tissue or a soft cloth.

If lamp does not light:

Remove diffuser. Check that lamp is secure; replace if necessary.

Call Rinn Customer Service with any questions.

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Rinn viewers utilize fluorescent lamps (bulbs). Fluorescent lamps contain mercury. Dispose of according to local, state or federal laws.

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WARRANTY

DENTSPLY RINN warrants this product to be free of defective materials and workmanship for one year from date of purchase when used in accordance with the operation instructions furnished herewith. This warranty is limited to the original purchaser. Any component which our examination reveals to be defective shall be either repaired or replaced at our option. There are no other warranties, either expressed or implied. Damages are limited to repair or replacement of product. DENTSPLY RINN expressly disclaims liability for incidental and consequential damages resulting from use of its product. DENTSPLY RINN neither assumes, nor authorizes any person to assume for it, any other liability in connection with the sale or use of its product.

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DENTSPLY
RINN

A Division of DENTSPLY International

1212 Abbott Drive
Elgin, IL 60123-1819 USA

Phone: 847 742 1115

Customer Service: 800 323 0970

FAX (24 hours a day): 800 544 0787

www.rinncorp.com

International:

+1 847 742-1140 Phone / Téléphone / Telefon / Telefono / Teléfono

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